

APESIN AP 3



Liquid disinfecting surface cleaner

- Material friendly
- Versatile
- Effective

Product Profile

- APESIN AP 3 disinfects and cleans in one operation, is effective against bacteria, levurocides and virucidal with limited effectiveness (HBV and HIV).
- APESIN AP 3 is economical in use, and it has excellent cleaning results.
- It is free of formaldehydes, phenols and chlorine-based disinfectants.
- APESIN AP 3 leaves a pleasant lime-scent.

Area of application

- Suitable for disinfection and cleaning of all water-proof surfaces and equipment throughout the hospital, as well as in sanatoriums, old people's homes and the doctor's office.
- Also suitable for usage in cosmetic and pharmaceutical industry.
- Always take care of complete wetting of the surfaces.

Ingredients:

Ingredients per 100g: 5,25g DIDECYLDIMONIUM CHLORIDE, 5 - <15% Cationic surfactants, <5% Non-ionic surfactants, Perfumes, LINALOOL, LIMONENE, GERANIOL

Application & Dosing




Dose in relation to application and degree of soil. Follow below instructions.




Disinfectant: suitable for areas with specific hygiene requirements.



Manual floor cleaning: Wipe the floor with a clean mop.

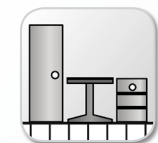

Surface cleaning: wipe the surface wet with a clean cloth.

	Exposure time			
	15 min.	30 min.	60 min.	240 min.
DGHH-VAH, low organic load	3%	2%	1%	0,5%
DGHH-VAH, high organic load			1,5%	0,75%
HBV (Hepatitis)	3%		2%	
HIV	3%		2%	

Sustainable development and production:



Product safety, storage and environmental instructions

Safety: For professional use only. For further information, see Safety Data Sheet. Use biocides safely. Always read the label and product information before use.

Storage: Store at room temperature in the original container.

Environmental: Only give completely emptied packaging to special waste disposer.

Sales units:

Order No. 404787 1 x 10 L

Your trusted partner on site:

pH-value:

